



Individual Test Action April 26, 2011

\*\*\*THE BELOW INFORMATION SENT IN THE MARCH LIVE WIRE HAS BEEN SUSPENDED, SO PLEASE DISREGARD THE CHANGES. WE APOLOGIZE FOR ANY INCONVENIENCE.\*\*\*

**NEW TESTS - Please update your EMR catalog with those appropriate to your practice**

<b>NEW TEST Available 04/25/2011</b>							
Order Test Code	Test Name	Mnemonic	Category/Type	Result Code	Result Name	Mnemonic	Ref Range - Units of Measure
1516550	INHIBITOR SCREEN	INHIB SCR	Group Parent Non-Orderable Child	1516725	INHIBITOR SCREEN DETAIL	XINHIB SCR	Neg Text
<b>Specimen Collection:</b> Please indicate anticoagulant therapy. Collect 6-8 Sodium Citrate 3.2% (Blue Top) tubes. Tubes must be filled to the proper level, no hemolysis. Improperly filled tubes can give erroneous results. Room Temperature whole blood specimens must be transported to lab immediately. If sending citrated plasma aliquots, they must be double spun then aliquot 1.5 ml plasma from each tube into individual plastic aliquot tubes and freeze. <b>Do not pool aliquots</b> - 5 to 6 individual aliquots are necessary for testing.		<b>CPT Code:</b> Multiple					

**INACTIVE TESTS - Please update your EMR catalog with those appropriate to your practice**

<b>INACTIVE Effective 04/25/2011</b>				
Order Test Code	Test Name	Mnemonic	Replacement Code	Replacement Name
1501650	INHIBITOR SCREEN	INHIB SCR	1516550	INHIBITOR SCREEN