

Package Insert for Personal Use Kit

**Read all package insert directions carefully before sample collection.
 Test results may be invalid if test is not performed properly.**




Sample Deposit

- 1.** Place supplied collection paper inside toilet bowl on top of water.
- 2.** Deposit stool sample on top of collection paper.
- 3.** Collect sample from stool before paper sinks and stool sample touches water.
- 4.** Flush. Collection paper is biodegradable and will not harm septic systems.

Sample Collection


- 1.** Fill in all required information on the sampling bottle.

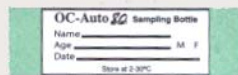

Open green cap by twisting and lifting.



- 2.** Scrape the surface of the fecal sample with the sample probe.

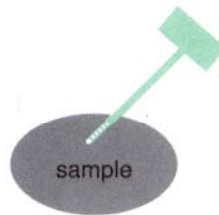

Cover the grooved portion of the sample probe completely with stool sample.



- 3.** Close sampling bottle by inserting the sample probe and snap green cap on tightly. Do not reopen.



Insert the sampling bottle in the Bio-Hazard baggie then mail it back to your ordering doctor or RML in the provided envelope.

~ Do Not mail to Polymedco. ~

