



**Test Name:** Homocystine Quantitative, Urine

**Reference Lab:** Arup

**Ref Lab Fax:**

**Test Code:** NEW TEST

**Ref Lab Phone:**

**Date of Search:** 01/17/2012

---

order as **NEW TEST**

footnote: **ARUP tc# 80413 Homocystine Quantitative, Urine**

specimen: **5mL(3mL) well mixed Urine from 24hr Collection FROZEN**  
(Random Urine is acceptable)

**Freeze Urine ASAP!**

**Please indicate Total Volume and Collection Duration on specimen Aliquot!**

Stability: Ambient= Unacceptable; Refrigerated= 24 hours; Frozen= 1 month

Note: **Patients should abstain from vitamin C ingestion for 48 hours prior to specimen collection.**

**Unacceptable Conditions:** pH less than 5 or greater than 8. Specimens with acid or other preservatives. Ascorbic acid interferes with this assay. Large amounts of blood or hemoglobin can interfere with quantitation.

Interpretive Data: For timed specimens other than 24 hours, the result will be extrapolated to represent a 24 hour time period. Reference ranges may not apply.

CPT Code(s): 82131

BILL CLIENT : \$105

BILL PATIENT : \$115

Send-out tests ordered as "NEW TEST"s are not built in the RML computer system and therefore Cpt Code(s) and Pricing may be subject to change by the performing laboratory.