

NEW TESTS - Please update your EMR catalog with those appropriate to your practice

New Test Effective : 3/2/2015

Orderable Test Code	Test Name	Mnemonic	Category/Type	Result Code	Mnemonic	Result Name	Reference Range	UOM
3657450	NIACIN (Vitamin B3)	NIACIN	Detail	3657450	NIACIN	NIACIN (Vitamin B3)	By Report	NG/ML

4mL(1mL) EDTA (lavender-top)-plasma.
 Separate specimens must be submitted when multiple tests are ordered
Plasma must be Frozen ASAP from Collection! (CRITICAL FROZEN!)
Protect from light during collection, storage, and shipment.
 CPT: 84591 Unacceptable Conditions:Thawed specimens or specimens not protected from light. Grossly hemolyzed or lipemic specimens.
 LOINC: 18244-4 Stability:Ambient: Unacceptable; Refrigerated: Unacceptable; Frozen: 2 months

New Test Effective : 3/2/2015

Orderable Test Code	Test Name	Mnemonic	Category/Type	Result Code	Mnemonic	Result Name	Reference Range	UOM
3000427	VITAMIN A (Retinol)	VIT A	Group	3000475	VITA RETI	VIT A (RETINOL)	By Report	MG/L
				3000625	VITA PALM	VIT A (PALMITATE)	By Report	MG/L
				3000775	VITA INTR	VIT A INTERPRETATION	By Report	

1mL(0.2mL) Serum SST or Plasma from Heparin (sodium or lithium)
 EDTA Plasma is also acceptable.
 CPT: 84590 Separate serum or plasma within 1 hour of collection. Keep Refrigerated
 LOINC: Protect from light during collection, storage, and shipment.
 VITA RETI 2923-1 Stability:After separation from cells: Ambient: Unacceptable; Refrigerated: 1 month; Frozen: 1 year
 VITA PALM 38496-6
 VITA INTR 48767-8

Component	Ref Range	Age
Vitamin A (Retinol)	0.18-0.50 mg/L	0-1 month
	0.20-0.50 mg/L	2 months-12 years
	0.26-0.70 mg/L	13-17 years
	0.30-1.20 mg/L	18 years and older
Retinyl Palmitate	0.00-0.10 mg/L	0-150 years

MODIFICATION TESTS - Please update your EMR catalog with those appropriate to your practice

Modify Test Effective : Immediately

Orderable Test Code	Test Name	Mnemonic	Category/Type
5564400	GLOMERULAR BASEMENT MEMBRANE ANTIBODY	GBM AB	Detail

CPT code changed from 86526 to 83520

Modify Test Effective : Immediately

Orderable Test Code	Test Name	Mnemonic	Category/Type
5513230	MYOSITIS PROFILE	MYOSI PROF	Detail

Specimen Stability Change
 New Stability RT=4 days; RF=7 days; FZ=30 days.

Modify Test Effective : Immediately

Orderable Test Code	Test Name	Mnemonic	Category/Type
4030005	PAIN MANAGEMENT DRUG PANEL A	XPM PNL - A	Group

Test Dropped from Panel
 Result detail MIDAZOL (430105) will no longer be resulted in this panel

Modify Test Effective : 1/5/2015

Orderable Test Code	Test Name	Mnemonic	Category/Type
4003230	NAPROXEN	NAPROXEN	Detail

Multiple Changes : Volume, Stability, Set Up
 Specimen volume: 1 ml (0.25 ml). Collection tubes remain the same
 Stability: Room Temp and Refrigerated 16 days; Frozen 9 months.
 Set-up: Tues, Thur TAT 4-6 days

Modify Test Effective : 1/5/2015

Orderable Test Code	Test Name	Mnemonic	Category/Type
4502125	MITOTANE (LYSODREN) SER/PLASMA	MITOTANE	Detail

Change in Specimen Volume and Stability
 Specimen volume - 1 ml (0.3 ml)
 Stability: Room Temperature, Refrigerated and Frozen 14 days.

Modify Test		Effective : 1/5/2015	
Orderable Test Code	Test Name	Mnemonic	Category/Type
3632070	DIURETIC SCREEN URINE	DIURET S U	Detail
Specimen Stability Change New Stability: Room Temperature 48 hours; Refrigerated and Frozen 28 days.			
Modify Test		Effective : 2/9/2015	
Orderable Test Code	Test Name	Mnemonic	Category/Type
2015000	APOLIPOPROTEIN A-1	APO A1	Group
Specimen Volume: Change to 1mL (0.5 mL)			
Multiple Changes Specimen Volume: Change to 1mL (0.5 mL) Patient Prep: 12-14 Hour Fasting is no longer required. Stability: Room Temperature 7days; Refrigerated 10days; Frozen 90 days			
Modify Test		Effective : 2/9/2015	
Orderable Test Code	Test Name	Mnemonic	Category/Type
2015050	APOLIPOPROTEIN B	APO B	
Specimen Volume: Change to 1mL (0.5 mL)			
Multiple Changes Specimen Volume: Change to 1mL (0.5 mL) Patient Prep: 12-14 Hour Fasting is no longer required. Stability: Room Temperature 7days; Refrigerated 10days; Frozen 90 days			
Modify Test		Effective : 2/9/2015	
Orderable Test Code	Test Name	Mnemonic	Category/Type
2014950	APOLIPOPROTEIN A1 and B	APO A1 B	
Specimen Volume: Change to 1mL (0.5 mL)			
Multiple Changes Specimen Volume: Change to 1mL (0.5 mL) Patient Prep: 12-14 Hour Fasting is no longer required. Stability: Room Temperature 7days; Refrigerated 10days; Frozen 90 days			
Modify Test		Effective : 2/9/2015	
Orderable Test Code	Test Name	Mnemonic	Category/Type
3611450	MURAMIDASE (Lysozyme)	MURAMIDASE	
Specimen Requirements: Change to 1ml (0.5ml min) from 2ml Stability: Room Temperature 24 hours; Refrigerated 15 days; Frozen 18 days Set-up: Mon-Sat TAT: 1-2 days after set-up. Reference range changed to 5.0-11.0 ug/ml from 7.0-15.0 ug/ml			
Modify Test		Effective : 2/9/2015	
Orderable Test Code	Test Name	Mnemonic	Category/Type
4006865	CLOZAPINE (Clozaril)	CLOZAPINE	Group
CPT code will actually be 80159			
Multiple Changes Preferred specimen: Red No Gel Serum. EDTA Plasma also acceptable. Heparin PlasmaSpecimens (lithium or sodium) are NOT acceptable. Specimen volume change to 1mL(0.4mL) Stability:Room Temperature 8 days; Refrigerated and Frozen 1 month Set-up VARIES TAT 4-11 days Reference range: <20 NG/ML CPT code will change from 83789 to 80299			
Modify Test		Effective : 3/2/2015	
Orderable Test Code	Test Name	Mnemonic	Category/Type
3604800	VITAMIN E (Tocopherol)	VITAMIN E	Group
New Reference Ranges			
Component		Ref Range	Age
Vitamin E (α -Tocopherol)		1.0-3.5 mg/L	0-1 month
		2.0-6.0 mg/L	2-5 months
		3.5-8.0 mg/L	6 months-1 year
		5.5-9.0 mg/L	2-12 years
		5.5-18.0 mg/L	13 years and older
Vitamin E (γ -Tocopherol)		0.0-6.0 mg/L	0-150 years
Specimen volumes change to 1mL(0.2mL) Spin, separate & refrigerate or freeze specimen with 1 hr of collection. Stability: Room Temperature n/a, Refrigerated 1 month, Frozen 1 year RT specimens are not acceptable Set-up S/U SUN-SAT TAT 2-4 days Protect from light during collection, storage, and shipment.			

Modify Test		Effective : 3/2/2015	
Orderable Test Code	Test Name	Mnemonic	Category/Type
3603700	VITAMIN C	VITAMIN C	Detail
<p>Reference Range Chages 23-114 umol/L</p> <p>Protect from light during collection, storage, and shipment.</p>			
<p>Multiple Changes Specimen Collection: GREEN (Li or Na) top no gel tube ON ICE Processing: Spin, separate & freeze within 1 hr of collection. Specimen CANNOT be shared with other tests. Specimen volumes change from 2ml(1ml) to 0.5ml(0.3ml). Stability: STABLE:Room Temperature and Refrigerated n/a; Frozen 1 year Set-up S/U SUN,TUES-THU,SAT TAT 2-8 days</p>			

Modify Test		Effective : 3/2/2015	
Orderable Test Code	Test Name	Mnemonic	Category/Type
3603665	VITAMIN B2 (Riboflavin)	VITAMIN B2	Detail
<p>Reference Range Chages 5-50 nmol/L</p> <p>Protect from light during collection, storage, and shipment.</p>			
<p>Multiple Changes Specimen Collection: GREEN (Li or Na) top no gel tube Processing: Spin, separate & freeze within 1 hr of collection. Specimen CANNOT be shared with other tests. Specimen volumes change from 2ml(0.5ml) to 1ml(0.5ml). Stability: Room Temperature and Refrigerated n/a; Frozen 1 month Set-up S/U SUN,WED,FRI TAT 2-7 days</p>			

Modify Test		Effective : 3/2/2015						
Orderable Test Code	Test Name	Mnemonic	Category/Type	Result Code	Mnemonic	Result Name	Reference Range	UOM
3600653	CAROTENE	CAROTENE	Detail	3600653	CAROTENE	CAROTENE (Beta Carotene)	60-200 µg/dL	UG/DL
<p>Change in Stability and Reference Ranges 2mL(0.6mL) Serum from SST or Red No Gel Clot Tube - Separate and Freeze within 1 hr of collection. Protect from light during collection, storage, and shipment. Critical Frozen. Separate specimens must be submitted when multiple tests are ordered. Stability After separation from cells: Ambient: Unacceptable; Refrigerated: 4 hours; Frozen: 1 month. Unacceptable Conditions: Any specimen other than serum. Refrigerated or room temperature specimens. Hemolyzed or icteric specimens.</p>								

INACTIVATED TESTS - Please update your EMR catalog with those appropriate to your practice

INACTIVE TESTS Effective : Immediately - Please update your EMR catalog with those appropriate to your practice

Test Code	Test Name	Mnemonic	Replacement Mnemonic	Test Code	Test Name
4003580	PLACIDYL	PLACIDYL	Discontinued		Discontinued

INACTIVE TESTS Effective : Immediately - Please update your EMR catalog with those appropriate to your practice

Test Code	Test Name	Mnemonic	Replacement Mnemonic	Test Code	Test Name
3655000	MELANOCYTE STIMULATING HORMONE	MSH	Discontinued		Discontinued

INACTIVE TESTS Effective : Immediately - Please update your EMR catalog with those appropriate to your practice

Test Code	Test Name	Mnemonic	Replacement Mnemonic	Test Code	Test Name
4012125	DANTROLENE	DANTROLENE	Discontinued		Discontinued

INACTIVE TESTS Effective : Immediately - Please update your EMR catalog with those appropriate to your practice

Test Code	Test Name	Mnemonic	Replacement Mnemonic	Test Code	Test Name
3651555	PENTOBARBITAL LEVEL	PENTOB LVL	Discontinued		Discontinued

INACTIVE TESTS Effective : 3/02/2015 - Please update your EMR catalog with those appropriate to your practice

Test Code	Test Name	Mnemonic	Replacement Mnemonic	Test Code	Test Name
2750000	NIACIN AND METABOLITES	NIACIN	NIACIN	3657450	See Information in the New Test Section

INACTIVE TESTS Effective : 3/02/2015 - Please update your EMR catalog with those appropriate to your practice

Test Code	Test Name	Mnemonic	Replacement Mnemonic	Test Code	Test Name
3603650	VITAMIN A	VIT A	VIT A	3000427	See Information in the New Test Section