

NEW TESTS - Please update your EMR catalog with those appropriate to your practice

New Test Effective : 06/11/2018

Test Code	Test Name	Mnemonic	Category/Type	LOINC	Result Code	Mnemonic	Result Name	Reference Range UOM	Result Type
6987003	MySwab Vaginosis Profile	MySwab Vag	Group						
CPT: 87481x2 87512, 87799	Collect Swab in APTIMA Multitest Swab collection kit. Place swab in Swab Specimen Transport Tube, break shaft off at scoreline then recap tube. Transport/Storage: Room Temperature Stability: Room Temperature 30 Days, Refrigerated 30 Days, Frozen: N/A Testing Schedule Mon-Fri / TAT 1-2 days after set-up			69568-4	6987012	Bact Vag Sw	Bacterial Vaginosis	Not Supportive	Alpha
				62460-1	6987010	Candida Sp Sw	Candida Species	Not Detected	Alpha
				69563-5	6987011	Candida Glab Sw	Candida Glabrata	Not Detected	Alpha

New Test Effective : 06/11/2018

Test Code	Test Name	Mnemonic	Category/Type	LOINC	Result Code	Mnemonic	Result Name	Reference Range UOM	Result Type
6987004	MySwab Vaginosis Profile + Trichomonas	MySwab Vag TV	Group						
CPT: 87481x2 87512, 87799 87661	Collect Swab in APTIMA Multitest Swab collection kit. Place swab in Swab Specimen Transport Tube, break shaft off at scoreline then recap tube. Transport/Storage: Room Temperature Stability: Room Temperature 30 Days, Refrigerated 30 Days, Frozen: N/A Testing Schedule Mon-Fri / TAT 1-2 days after set-up			69568-4	6987012	Bact Vag Sw	Bacterial Vaginosis	Not Supportive	Alpha
				62460-1	6987010	Candida Sp Sw	Candida Species	Not Detected	Alpha
				69563-5	6987011	Candida Glab Sw	Candida Glabrata	Not Detected	Alpha
				69937-1	6910157	Trichomonas Sw	Trichomonas Sw	Not Detected	Alpha

New Test Effective : 06/11/2018

Test Code	Test Name	Mnemonic	Category/Type	LOINC	Result Code	Mnemonic	Result Name	Reference Range UOM	Result Type
6987005	MySwab Vaginosis Profile + STD	MySwab Vag STD	Group						
CPT: 87481x2 87512, 87799 87661, 87491 87591	Collect Swab in APTIMA Multitest Swab collection kit. Place swab in Swab Specimen Transport Tube, break shaft off at scoreline then recap tube. Transport/Storage: Room Temperature Stability: Room Temperature 30 Days, Refrigerated 30 Days, Frozen: N/A Testing Schedule Mon-Fri / TAT 1-2 days after set-up			69568-4	6987012	Bact Vag Sw	Bacterial Vaginosis	Not Supportive	Alpha
				62460-1	6987010	Candida Sp Sw	Candida Species	Not Detected	Alpha
				69563-5	6987011	Candida Glab Sw	Candida Glabrata	Not Detected	Alpha
				69937-1	6910157	Trichomonas Sw	Trichomonas Sw	Not Detected	Alpha
				35729-3	6910149	Chlamydia Sw	Chlamydia Sw	Not Detected	Alpha
				24111-7	6910153	Gonorrhea Sw	Gonorrhea Sw	Not Detected	Alpha

New Test Effective : 06/11/2018

Test Code	Test Name	Mnemonic	Category/Type	LOINC	Result Code	Mnemonic	Result Name	Reference Range UOM	Result Type
6987006	MySwab STD Profile	MySwab STD	Group						
CPT: 87529x2 87661, 87491 87591	Collect Swab in APTIMA Multitest Swab collection kit. Place swab in Swab Specimen Transport Tube, break shaft off at scoreline then recap tube. Transport/Storage: Room Temperature Stability: Room Temperature 30 Days, Refrigerated 30 Days, Frozen: N/A Testing Schedule Mon-Fri / TAT 1-2 days after set-up			69937-1	6910157	Trichomonas Sw	Trichomonas Sw	Not Detected	Alpha
				35729-3	6910149	Chlamydia Sw	Chlamydia Sw	Not Detected	Alpha
				24111-7	6910153	Gonorrhea Sw	Gonorrhea Sw	Not Detected	Alpha
				16130-7	6987013	HSV 1 Sw	Herpes Simplex Virus 1	Not Detected	Alpha
				16131-5	6987014	HSV 2 Sw	Herpes Simplex Virus 2	Not Detected	Alpha

New Test Effective : 06/11/2018									
Test Code	Test Name	Mnemonic	Category/Type	LOINC	Result Code	Mnemonic	Result Name	Reference Range UOM	Result Type
6987002	HSV 1 and 2 - Swab	HSV Swab	Group	16130-7	6987013	HSV 1 Sw	Herpes Simplex Virus 1	Not Detected	Alpha
				16131-5	6987014	HSV 2 Sw	Herpes Simplex Virus 2	Not Detected	Alpha
CPT: 87529x2	Collect Swab in APTIMA Multitest Swab collection kit. Place swab in Swab Specimen Transport Tube, break shaft off at scoreline then recap tube. Alternate Swab are the acceptable transport media: VTM, UTM, M4RT, M4 and M5 media. Alternate Swab Must Keep Refrigerated Alternate Swabs should be received promptly so they can be transferred to Aptima STM within 72 hrs of collection. Transport/Storage: Room Temperature / Refrigerated Stability: Room Temperature 30 Days, Refrigerated 30 Days, Frozen: N/A Testing Schedule Mon-Fri / TAT 1-2 days after set-up								
New Test Effective : 06/11/2018									
Test Code	Test Name	Mnemonic	Category/Type	LOINC	Result Code	Mnemonic	Result Name	Reference Range UOM	Result Type
6987000	Candida Vaginitis - Swab	Cand Vag Swab	Group	62460-1	6987010	Candida Sp Sw	Candida Species	Not Detected	Alpha
				69563-5	6987011	Candida Glab Sw	Candida Glabrata	Not Detected	Alpha
CPT: 87481x2	Collect Swab in APTIMA Multitest Swab collection kit. Place swab in Swab Specimen Transport Tube, break shaft off at scoreline then recap tube. Transport/Storage: Room Temperature Stability: Room Temperature 30 Days, Refrigerated 30 Days, Frozen: N/A Testing Schedule Mon-Fri / TAT 1-2 days after set-up								
New Test Effective : 06/11/2018									
Test Code	Test Name	Mnemonic	Category/Type	LOINC	Result Code	Mnemonic	Result Name	Reference Range UOM	Result Type
6987001	Bacterial Vaginosis - Swab	Bact Vag Swab	Detail	69568-4	6987012	Bact Vag Sw	Bacterial Vaginosis	Not Supportive	Alpha
CPT: 87512 87799	Collect Swab in APTIMA Multitest Swab collection kit. Place swab in Swab Specimen Transport Tube, break shaft off at scoreline then recap tube. Transport/Storage: Room Temperature Stability: Room Temperature 30 Days, Refrigerated 30 Days, Frozen: N/A Testing Schedule Mon-Fri / TAT 1-2 days after set-up								

The following tests are components of the new MySwab group tests and are also currently available to be ordered as stand-alone tests. They have been included on this Livewire for your convenience.

New Test Effective : Currently Available									
Test Code	Test Name	Mnemonic	Category/Type	LOINC	Result Code	Mnemonic	Result Name	Reference Range UOM	Result Type
6910109	C.trachomatis Swab	CT Swab	Detail	35729-3	6910149	Chlamydia Sw	Chlamydia Sw	Not Detected	Alpha
CPT: 87491	Collect Swab in APTIMA Multitest Swab collection kit. Place swab in Swab Specimen Transport Tube, break shaft off at scoreline then recap tube. Transport/Storage: Room Temperature Stability: Room Temperature 30 Days, Refrigerated 30 Days, Frozen: N/A Testing Schedule Mon-Fri / TAT 1-2 days after set-up								
New Test Effective : Currently Available									
Test Code	Test Name	Mnemonic	Category/Type	LOINC	Result Code	Mnemonic	Result Name	Reference Range UOM	Result Type
6910115	N.gonorrhoeae Swab	GC Swab	Detail	24111-7	6910153	Gonorrhea Sw	Gonorrhea Sw	Not Detected	Alpha
CPT: 87591	Collect Swab in APTIMA Multitest Swab collection kit. Place swab in Swab Specimen Transport Tube, break shaft off at scoreline then recap tube. Transport/Storage: Room Temperature Stability: Room Temperature 30 Days, Refrigerated 30 Days, Frozen: N/A Testing Schedule Mon-Fri / TAT 1-2 days after set-up								
New Test Effective : Currently Available									
Test Code	Test Name	Mnemonic	Category/Type	LOINC	Result Code	Mnemonic	Result Name	Reference Range UOM	Result Type
6910121	Trichomonas Swab	Trich Swab	Detail	69937-1	6910157	Trichomonas Sw	Trichomonas Sw	Not Detected	Alpha
CPT: 87661	Collect Swab in APTIMA Multitest Swab collection kit. Place swab in Swab Specimen Transport Tube, break shaft off at scoreline then recap tube. Transport/Storage: Room Temperature Stability: Room Temperature 30 Days, Refrigerated 30 Days, Frozen: N/A Testing Schedule Mon-Fri / TAT 1-2 days after set-up								




A symptom-specific approach to diagnosis of women's genital tract infections

MySwab is powered by robust and reliable nucleic acid amplification methodology that you can trust. Our test menu is specifically designed to target the most common genital pathogens affecting women's health and sexually transmitted diseases, evaluated on a single Aptima swab.

The MySwab approach is comprehensive, cost effective and reliable.

Simplify collection with a single swab

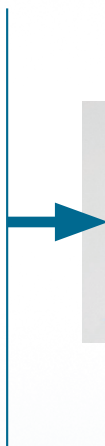

Bacterial Vaginosis

- Gardnerella vaginalis
- Lactobacillus species

Candida Vaginitis

- C. albicans
- C. glabrata
- C. krusei
- C. tropicalis

Sexually Transmitted Pathogens

- Chlamydia trachomatis
- Neisseria gonorrhoeae
- Trichomonas vaginalis
- Herpes Simplex Virus 1 and 2
- Mycoplasma genitalium (available soon!)



Test menu

Test name	Clinical presentation	Components	Test number	Specimen and stability
	Symptoms of vaginosis, including discomfort and discharge, with low risk for sexually transmitted infections	Bacterial Vaginosis Candida Vaginitis	6987003	Vaginal swab collected with the APTIMA collection kit Stability : Refrigerated = 30 days Transport at Room Temp
	Symptoms of vaginosis, including discomfort and discharge, and at risk for coinfection with Trichomonas	Bacterial Vaginosis Candida Vaginitis Trichomonas Vaginalis	6987004	
	Symptoms of vaginosis, including discomfort and discharge, and at risk for coinfection with Trichomonas, Chlamydia and/or Gonorrhea	Bacterial Vaginosis Candida Vaginitis Trichomonas Vaginalis Chlamydia Trachomatis Neisseria Gonorrhoeae	6987005	
	Patients with pain and/or visible lesions and potential for coinfection with other sexually transmitted infections	Trichomonas Vaginalis Chlamydia Trachomatis Neisseria Gonorrhoeae Herpes Simplex Virus 1 ** Herpes Simplex Virus 2 **	6987006	
	Option to order test components a la carte	Candida Vaginitis	6987000	
		Bacterial Vaginosis	6987001	
		Trichomonas Vaginalis	6910121	
		Chlamydia Trachomatis	6910109	
		Neisseria Gonorrhoeae	6910115	
		Herpes Simplex Virus 1 **	6987002	
		Herpes Simplex Virus 2 **		

**This assay is only FDA approved for clinician-collected swab specimens from anogenital lesions. Swab collections from other locations or in absence of visible lesions are considered an off label use and have not been validated at this laboratory. Results should not be used as the sole means for diagnosis and should be correlated with the clinical presentation.

