

SPECIMEN COLLECTION CHART

All specimens must be labeled with a first and last name and a second identifier, e.g., date of birth.




Pap/HPV Testing: ThinPrep

- HPV Only
- HPV Only Reflex to Genotype
- Pap 1: Pap Test
- Pap 2: Pap Test Reflex to HPV if ASCUS
- Pap 3: Pap Test and HPV
- Pap 4: Pap Test w/ HPV, Reflex to Genotypes 16, 18/4
- Pap 5: Pap Test Reflex to HPV if ASCUS or AGUS
- Pap 6: Pap Test Reflex to HPV if ASCUS or AGUS, Reflex to Genotype 16, 18/45
- Pap 7: Pap Test Reflex to HPV if ASCUS, ASC-H, or AGUS
- Pap 8: Pap Test Reflex to HPV if ASCUS, ASC-H, or AGUS, Reflex to Genotype 16, 18/45
- Pap 9: Pap Test Reflex to HPV if ASCUS, Reflex to Genotype 16, 18/45
- Pap 10: Pap Test, ThinPrep, Image Guided, Reflex to HPV if ASCUS, ASC-H, AGUS, or LSIL
- Pap 11: Pap Test, ThinPrep, Image Guided, Reflex to HPV if ASCUS, ASC-H, AGUS, or LSIL, Reflex Genotype 16, 18/45

Room Temp

Myswab

- Bacterial Vaginosis - Swab
- Candida Vaginitis - Swab
- Chlamydia trachomatis - Swab
- Herpes Simplex Virus 1 and 2 (HSV) - Swab
- MySwab STD Profile - Swab
- MySwab Vaginosis Profile + STD Profile - Swab
- MySwab Vaginosis Profile + Trichomonas - Swab
- MySwab Vaginosis Profile - Swab
- Neisseria gonorrhoeae - Swab
- Trichomonas - Swab

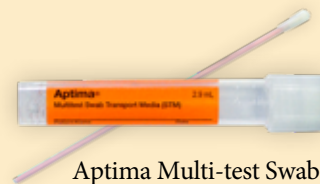

Aptima Multi-test Swab

*5 tests performed per swab *

Refrigerated

Vaginal/Genital Cultures:

Genital/Vaginal Cultures, Group B Streptococcus

Specimen Options:

- | | |
|---|--|
| White Copan Eswab (Preferred) <i>Break swab inside of container and replace cap</i> | Green Copan Eswab "Flexible minitip" (Alternative) <i>Break swab inside of container and replace cap</i> |
|---|--|



White Copan Eswab

Green Copan Eswab

Room Temp

Vaginitis Testing:



BD Affirm Ambient Temperature Transport Swabs (ATTS)

Testing includes:

- Gardnerella
- Candida (7 species)
- Trichomonas

Room Temp

Vaginal Profile Swab, no pH



White Copan Eswab (Preferred)

Green Copan Eswab
"Flexible minitip" (Alternative)

Testing includes:

- Trichomonas Antigen
- Wet Prep for Yeast
- Whiff Test
- Gram Stain for Bacterial Vaginosis
- Interpretation

Room Temp

Vaginal Profile Saline, with pH, Outreach



Sterile container with 1.5 mLs of saline


Testing includes:

- Trichomonas Antigen
- Wet Prep for Yeast
- pH
- Whiff Test
- Gram Stain for Bacterial Vaginosis
- Interpretation

Room Temp

STI Testing by Molecular: Chlamydia, Neisseria Gonorrhoeae, and Trichomonas: by Molecular

Specimen Options:

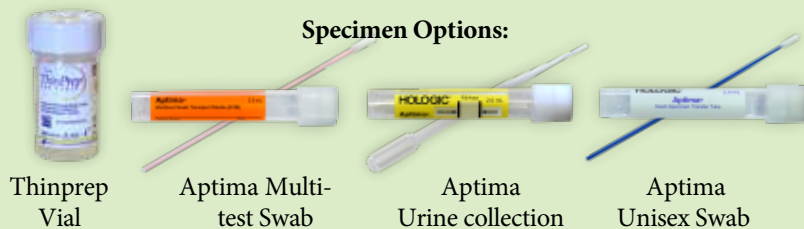

ThinPrep Vial


Aptima Multi-test Swab

Aptima Urine collection

Aptima Unisex Swab

Room Temp

STI- Herpes Simplex Virus 1 and 2 (HSV) Swab - TMA DNA, PCR

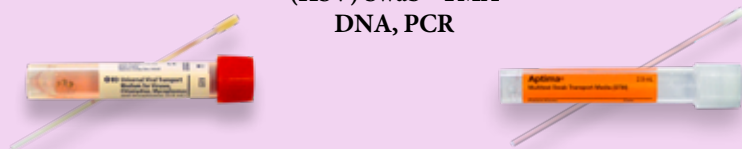

UTM Mycoplasma genitalium

Refrigerated

Aptima Multi-test Swab

