

**INACTIVATED TESTS - Please update your EMR catalog with those appropriate to your practice**

**INACTIVE Effective : 8/13/2018**

Inactive	Inactive	Inactive	Replacement Information			
Test Code	Test Name	Mnemonic	Test Code	Mnemonic	Test Name	
6905929	HPV DNA, High Risk, Anal-Rectal	HPV Hi Rect	6905701	HPV Hi/Lo Rect	HPV DNA, High and Low Risk, Anal-Rectal	

*Replacement test already exists... information below*

Test Code	Test Name	Mnemonic	Category/Type	LOINC	Result Code	Mnemonic	Result Name	Reference Range UOM	Result Type
6905701	HPV DNA, High and Low Risk, Anal-Rectal	HPV Hi/Lo Rect	Group	56140-7	6905703	HPV DNA High R	HPV DNA (High Risk)	Not Detected	Alpha
				50642-8	6905705	HPV DNA Low R	HPV DNA (Low Risk)	Not Detected	Alpha

CPT: 87624

Preferred Specimen: Dacron swab or anal-rectal brush collected in Digene Standard Transport Medium (STM).

**Collection Instructions**

To collect an anal-rectal sample, a tap water-moistened Dacron swab is used. The Dacron swab is inserted about 5-6 cm into the anal canal past the anal verge, into the rectal vault. This is done without direct visualization of the anal canal. Firm lateral pressure is applied to the swab handle as it is rotated and slowly withdrawn from the anal canal, inscribing a cone-shaped arc. Avoid using cotton swabs on a wooden stick because the handle may break and splinter during collection.

Specimen Stability: Room temperature 14 days, Refrigerated 21 days, Frozen 90 days

Reject Criteria: Specimens other than Digene® (STM) specimens • Liquid based pap smear sample

Set up: Tues-Sat; Report available: 3-5 days