

INACTIVE Effective : 03/16/2020									
Inactive	Inactive	Inactive	Replacement Information						
Test Code	Test Name	Mnemonic	Test Code	Mnemonic	Test Name				
4275001	Kappa/Lambda Light Chains w/ Pathologist interpretation	FREE LIGHT	6907201	Free Light Only	Kappa/Lambda Light Chains without Interpretation				
Build information for this replacement test has been added below.									
This test code 4275001 is being discontinued in favor of the Kappa/Lambda Light Chains without Interpretation.									
Test Code	Test Name	Mnemonic	Category/Type	LOINC	Result Code	Mnemonic	Result Name	Reference Range UOM	Result Type
6907201	Kappa/Lambda Light Chains without Interpretation	Free Light Only	Group	36916-5	5006125	Kappa Light	Kappa Light Chains	3.30-19.40 mg/L	Numeric XXXXX.XX
CPT: 83883x2	This is the replacement code for the Deactivated 4275001 Specimen processing: 2mL serum, clot Activator SST, refrigerated Instructions: Allow specimen to clot completely at room temperature. Separate serum or plasma from cells ASAP or within 2 hours of collection. Stability after separation from cells: Ambient - 4hours; Refrigerated - 7 days; Frozen - 1 month (avoid repeated freeze/thaw cycles). Set up: Mon -Fri. TAT: 1-3 day			33944-0	5006100	Lambda Light	Lambda Light Chains	5.71 - 26.30 mg/L	Numeric XXXXX.XX
				48378-4	5006150	Kappa/Lambda	Kappa/Lambda Ratio	0.26 - 1.65 Ratio	Numeric XXXXX.XX